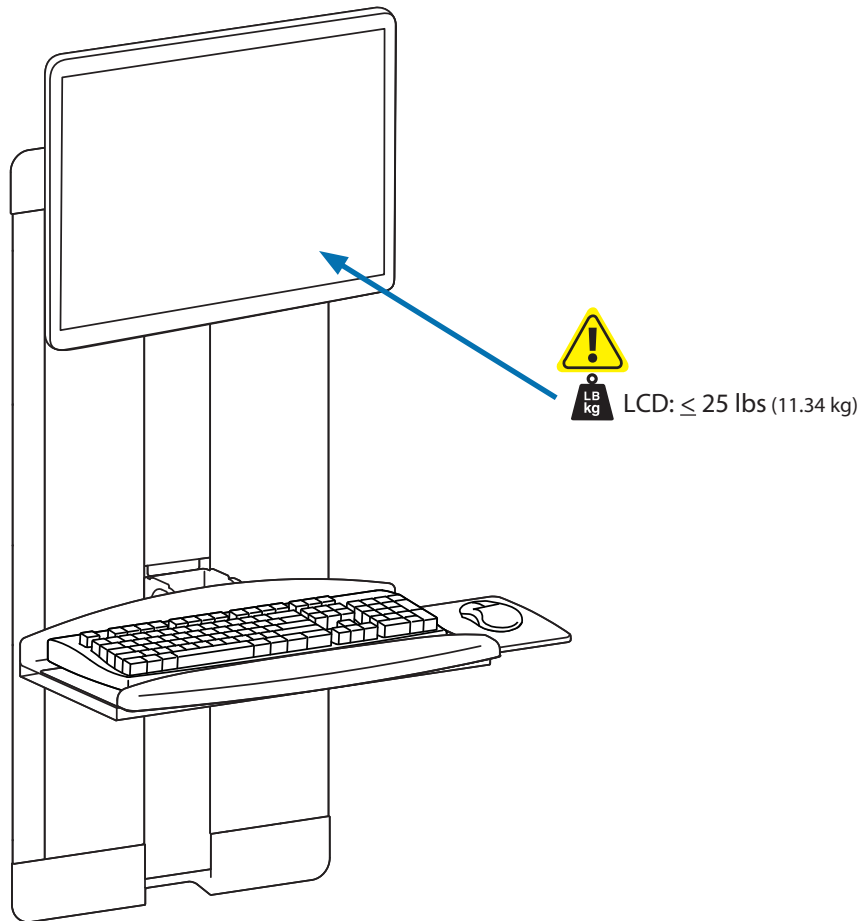
ergotron®

StyleView Vertical Lift Hallway

EN User Guide

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www.ergotron.com | USA: 1-800-888-8458 | Europe: +31 (0)33-45 45 600 | China: 400-120-3051 | Japan: japansupport@ergotron.com

Components
Onderdelen

Componentes
Componenti

Composants
Komponenter

Komponenten
構成部品

CAUTION: Do Not remove cardboard spacer until instructed to do so.
 PRECAUCIÓN: No retire el separador de cartón hasta que lo indiquen las instrucciones.
 ATTENTION: NE PAS retirer l'espaceur en carton tant qu'il ne vous est pas demandé de le faire.
 VORSICHT: Entfernen Sie die Abstandshalter aus Pappe erst, wenn Sie dazu aufgefordert werden.
 ATTENZIONE: non rimuovere il distanziale in cartone finché non si arriva all'istruzione specifica.
 VARNING: Ta inte bort avskiljaren av kartong förrän du anvisas att göra detta.
 注意: 段ボールのスペーサーは指示があるまで取り外さないでください。
 小心: 除非要求, 否则请勿取下纸垫片。
 주의: 별도의 지시가 있기 전까지는 카르보드 스페이서를 제거하지 마십시오.

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$$A = 16.5 - (B + 3.5)$$

$$B = \leq 7-1/4" (184 \text{ mm})$$

Minimizing Vertical Lift Storage Profile

For VL Series to remain less than 4" (102 mm)* from the wall* when in storage position, follow these guidelines:

1. Monitor must not be more than 2-1/2" (63 mm) thick.
2. Keyboard should not be more than 7-1/4" (184 mm) deep.
3. Dimension "A" must be no greater than 8-3/4" (222 mm).
4. Use Standard Keyboard Tray option only (Slide-Out Keyboard Tray option depth \geq 4-1/2", 114 mm).

* Exceptions: If you want to install your keyboard in a forward position while remaining \leq 4" (102 mm) from the wall, "A" \leq 7" (178 mm). See Figure 1.

If you want to use a keyboard wrist rest, dimension "A" can be calculated by adding 3-1/2" (89 mm) to the keyboard depth "B" and subtracting this sum from 16-1/2" (419 mm). See Figure 2.

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
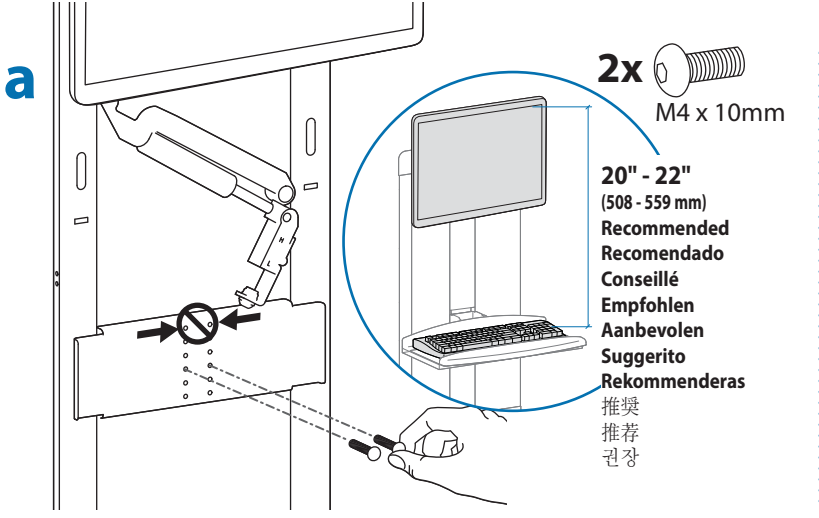
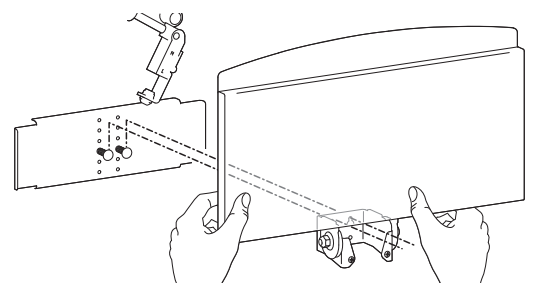
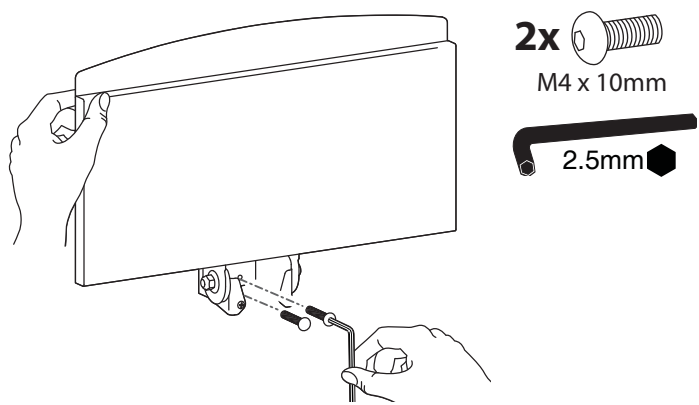
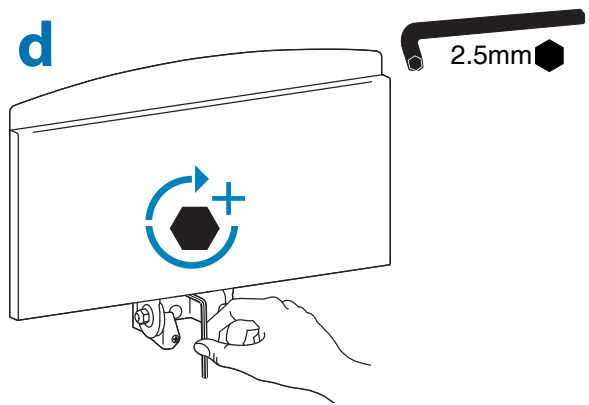
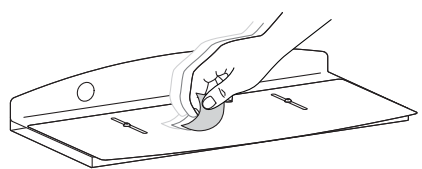
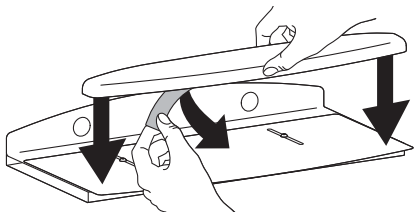
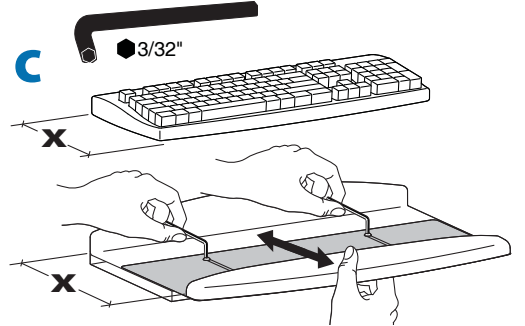
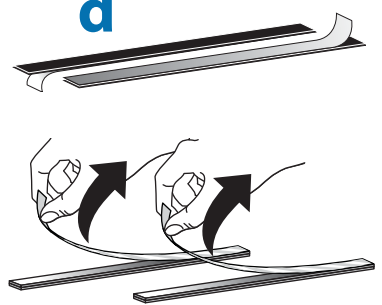
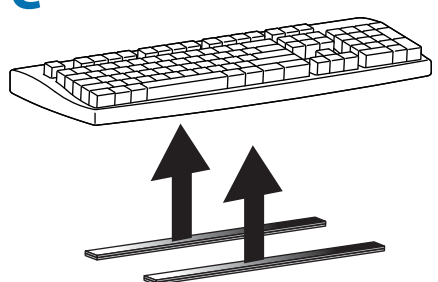
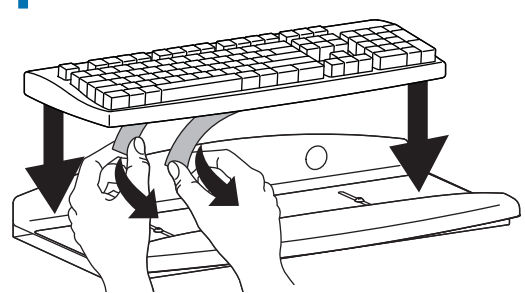
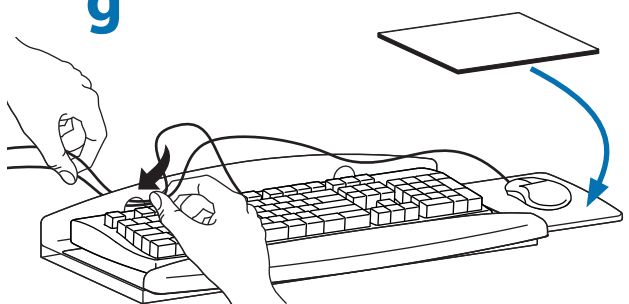
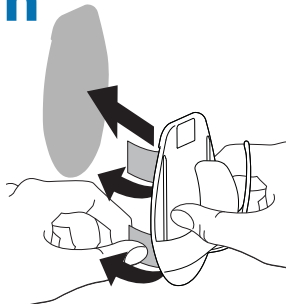
i

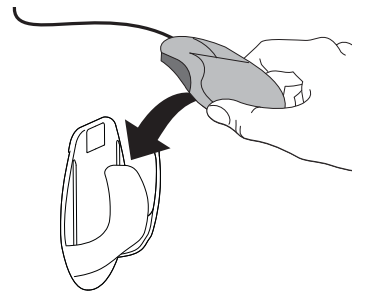8

It is important that you adjust this product according to the weight of the mounted equipment as described in the following steps. Any time equipment is added or removed from this product, resulting in a change in the weight of the mounted load, you should repeat these adjustment steps to ensure safe and optimum operation.

Adjustments should move smoothly and easily through the full range of motion and stay where you set it. If adjustments are difficult and do not stay in the desired position, follow the instructions to loosen or tighten the tension to create a smooth, easy adjustment motion. Depending on your product and the adjustment, it may take several turns to notice a difference.

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© 2019 Ergotron, Inc. All rights reserved. StyleView™ is a registered trademark of Ergotron, Inc.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.